
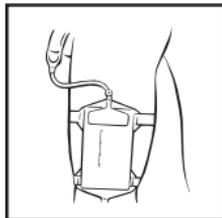
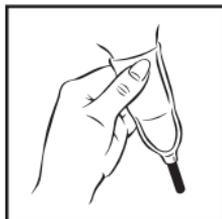


# Fitting the sheath

# BARD

The sheath is a strapless penile sheath: it has an adhesive coating inside to ensure a safe and simple procedure for fitting.

- If necessary trim pubic hair.
- Clean and dry pubic area.
- Remove sheath from its wrapping.
- Place rolled end over the end of the penis, leaving a small space between the end of the penis and the cup of the sheath. If uncircumcised, do not retract the foreskin but allow it to remain over the glans. Avoid contact between the adhesive and the glans.
- Slowly unroll the sheath along the shaft of the penis, then gently squeeze the sheath around the penis to ensure even adhesion. Try to avoid leaving a rolled 'collar' of sheath around the base of the penis.
- Finally, connect the urine collection bag. Always check that the connections are secure before use.
- Wear time will vary from user to user. It is recommended that a sheath be changed every 24 hours for reasons of hygiene.
- To remove: The sheath may be removed by gently rolling it off the penis. Avoid simply pulling the sheath off. Removing the sheath whilst having a bath or shower may increase comfort.

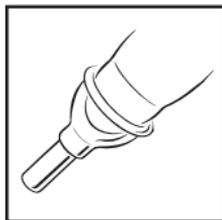

**Do not use on irritated or compromised skin.  
Discontinue use if irritation occurs.**

  
CAUTION,  
consult  
accompanying  
documents

  
Use By

  
Non-Sterile

  
Single Use

  
Manufacturer

  
Units

  
Authorized  
Representative

  
Not made  
with natural  
rubber latex

 **Manufacturer:**  
C. R. Bard, Inc.  
Covington, GA 30014 USA  
(800) 526-4455  
www.bardcare.com

 Bard Limited, Forest House  
Crawley, West Sussex UK  
RH11 9BP  
+44 1293 527 888

Bard is a trademark and/or  
registered trademark of  
C. R. Bard, Inc.  
©2016 C. R. Bard, Inc.  
All rights reserved.

PK2450033 11/2016