

## THANK YOU FOR USING THE VERTALOC® WRIST BRACE WITH THUMB SPICA

(MODEL# USD301WWT)

Your VertaLoc® Wrist Brace with Thumb Spica has been designed to maximize support and enhance comfort while allowing breathability.

YOU HAVE RECEIVED:

- ONE (1) WRIST BRACE COMPLETE WITH CLOSURE SYSTEM



## FIT YOUR BRACE IN 4 EASY STEPS



1. OPEN ALL STRAPS. INSERT YOUR HAND INTO THE BRACE AND PLACE THUMB INTO THE THUMB SECTION.



2. FASTEN THE THUMB STRAP SECURELY. NEXT, FASTEN THE OTHER THREE (3) STRAPS BY WRAPPING EACH STRAP AROUND YOUR WRIST AND THROUGH THE "D" RING. FASTEN SECURELY.



3. TIGHTEN ALL STRAPS UNTIL SNUG AND COMFORTABLE. ADJUSTMENTS MAY BE NECESSARY AS HAND, THUMB AND WRIST SWELLING PATTERNS CHANGE.



4. TO IMPROVE COMFORT, STRAPS MAY BE TRIMMED WITH SCISSORS.

*\*Adjustment of the aluminum thumb and palmar supports can be performed by a certified medical professional.*

## CARING FOR YOUR VERTALOC® WRIST BRACE WITH THUMB SPICA:

Remove aluminum supports before washing. Hand wash in warm water using mild soap.

Rinse thoroughly by hand, line dry only.

Do not machine wash or dry. Do not expose to direct heat sources such as heaters, stoves, and radiators.

## VERTALOC® LITE WRIST BRACE WARRANTY:

✓ YOUR VERTALOC® WRIST BRACE WITH THUMB SPICA IS WARRANTED TO BE FREE OF MATERIAL OR WORKMANSHIP DEFECTS.

✓ ANY DEFECTIVE COMPONENTS OF YOUR VERTALOC® WRIST BRACE MAY BE REPLACED OR CREDITED AT THE DISCRETION OF VERTALOC, INC.

✓ THIS PRODUCT IS WARRANTED FOR ONE YEAR FROM THE DATE OF PURCHASE.

**QUESTIONS OR COMMENTS? WE ARE ALWAYS HERE TO HELP YOU.  
CALL US TOLL FREE 855-VERTALOC (855-837-8256)**